


Detects IgG & IgM anti-SARS-CoV-2 in 10 minutes

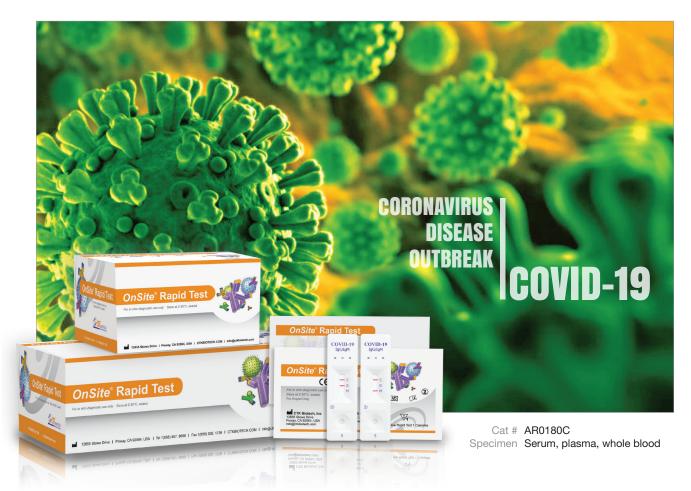



PRODUCT FEATURES

- Whole blood specimen allows testing at point-of-care, saving crucial time.
- Results in 10 minutes allow doctors to take immediate action.
- ▶ 96.9% sensitivity, 99.4% specificity (vs. PCR-confirmed specimens)
- Complementary to PCR by removing false negatives caused by specimen collection issue
- Compatible with Rapid Test Reader (RTR-1)
 - 200 tests per hour in batch mode
 - Capable of connecting to Laboratory Information System (LIS)
 - Built-in barcode reader provides faster test set up
 - Store up to 1,000 results, presented on screen, via printout, or uploaded to database













COVID-19 IgG/IgM Rapid Test (€



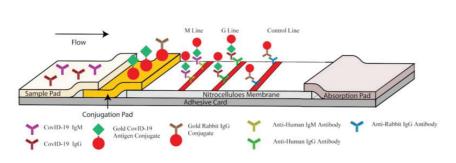

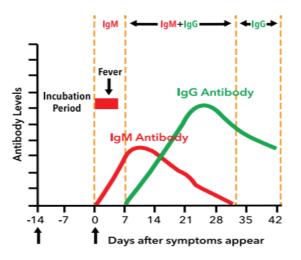

TEST PRINCIPLE

COVID-19 INFECTION CHART

COVID-19

NEED OF RAPID DIAGNOSTIC TEST FOR COVID-19

- Long Turn around Time, complicated in operation of RT-PCR and other methods.
- PCR Test required certified laboratories, expensive technique
- and trained technicians to operate.
- There are some number of false negatives in RT-PCR of COVID-19. (Gallagher et al., Feb 2020)
- Hence, there is an urgent need for a rapid, simple to ease, sensitive and accurate test to quickly identify infected patients of SARS-CoV-2

CLINICAL PERFORMANCE

	COVID-19 IgG/IgM Rapid Test IgG Results		COVID-19 IgG/IgM Rapid Test IgM Results	
PCR Test	Positive	Negative	Positive	Negative
Positive	216	7	174	49
Negative	0	328	2	326
Total	216	335	176	375

Relative IgG Sensitivity: 96.86% (95% CI: 93.66%-98.47%), Relative IgG Specificity: 100% (95% CI: 98.84%-100%) Relative IgM Sensitivity: 78.03% (95% CI: 72.14%-82.96%), Relative IgM Specificity: 99.39% (95% CI: 97.80%-99.83%) Relative Test Sensitivity: 96.86% (95% CI: 93.66%-98.47%), Relative Test Specificity: 99.39% (95% CI: 97.80%-99.83%) Overall Agreement: 98.37% (95% CI: 96.93%- 99.14%)





American Technology

For India

www.athenesedx.com | info@athenesedx.com | +91-44-22541131 Module No. 407 & 408, 4th Floor, TICEL Bio Park II, No. 5, CSIR Road Taramani, Chennai - 600113, INDIA